



**Libral Traders Pvt. Ltd.**  
*Your smile. Our care.*

## **Cleaning & Sterilization Instructions for**

### **All Plastic Products (Cheek Retractors, Impression Trays, etc..)**

#### **General Instructions**

**Caution :** Read the following instructions properly before using our products

Our Cheek Retractors, Impression Trays are Made of High Quality Materials & Does not requires any special handling procedure.

#### **Cleaning**

Immediately after use, thoroughly rinse fluids / debris from the device with copious amounts of cool running water. Clean with soap and water. Visually check to ensure surfaces are clean and free from damage, stains and bioburden. Rinse Retractors with distilled or de-mineralized water if possible. Inspect device for visual damage, wear and tear or deterioration. Discard broken, defective or worn-out products.

#### **Autoclave (Steam Sterilization)**

The preferred manner of sterilization is Autoclaving or Steam Sterilization. We recommend that you autoclave on the "plastics" setting of your autoclave, or as mentioned on the Product page of our Catalogue. In any event, the temperature should not exceed 289 F (143 C). Because there are many types of autoclaves in use, it is difficult to provide guidance on the specific instructions whenever possible. In, general, sterilize wrapped items for 30 minutes and unwrapped items for 20 minutes at 250 F (121 C). (Do not begin timing until the autoclave reaches the desired temperature) After sterilization, retractors should be kept in their sealed package and opened only at point of use.

#### **Cold Sterilization**

If you do not have an autoclave, or it is not convenient to use it, you can cold sterilize using solutions containing glutaraldehyde, low concentrations of hydrogen peroxide, isopropyl alcohol or other cold sterilization containing mostly inert ingredients.

#### **Dry Heat or Chemclave Sterilization**

Dry heat and chemclave sterilization are not acceptable for use on the Retractors. The temperature of these methods are too extreme and will result in the melting of the plastics.

*For More Information contact us :*

**Libral Traders Pvt. Ltd. B-84/1, Okhla Industrial Area Phase 2, New Delhi 110020, India**  
**Customer Care email [info@libraltraders.com](mailto:info@libraltraders.com) tele +91-11-40726678**